


# IGENITY<sup>®</sup> TISSUE SAMPLE INSTRUCTIONS


*PLEASE READ KIT INSTRUCTIONS COMPLETELY BEFORE PERFORMING TEST.*




Call 855.324.9774 to order, or visit [NEOGEN.com](http://NEOGEN.com)

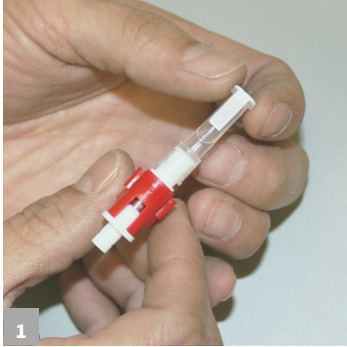

Remove TSU punch from the packaging. Punches should be assembled as one piece; if they have come apart, reassemble. Align the groove of the punch with the chamber at the base of the applicator.

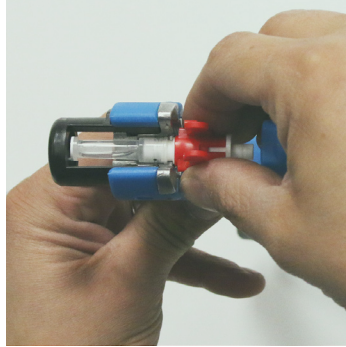


Rotate the black chamber to lock the punch into the device. Carefully squeeze the applicator handles together, guiding the punch tip into place if necessary. When the gun bolt rests flush against the red plastic clip, release the handle.

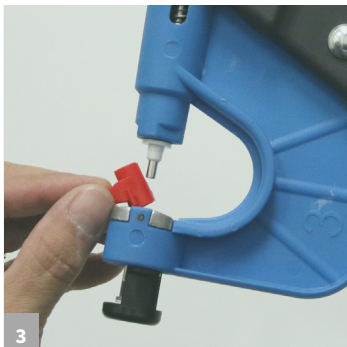

Remove the red plastic clip by pulling it outward. Be careful as the metal cutter above the clip is very sharp. Ensure the applicator is loaded with an unused green punch (if the red plunger is visible, the punch is used and should be replaced with one that is unused).



Slide the applicator over the animal's ear and position the metal cutter one inch from the edge of the ear, making sure to avoid any obvious veins or ridges. Squeeze handles together to take a sample and then release to free the ear. Try to do this in one swift, fluid, motion.

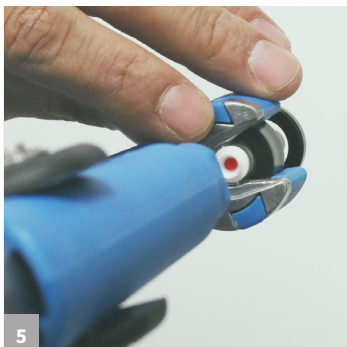

Reopen the chamber and remove the punch from the applicator. Check that sampling has been successful (red plunger is visible, sample is in fluid in punch). Remove the used cutter from the applicator by pulling the handles apart. Discard carefully as the cutter is very sharp.

